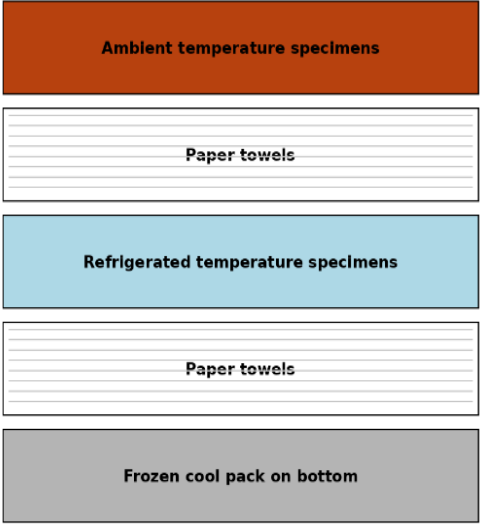
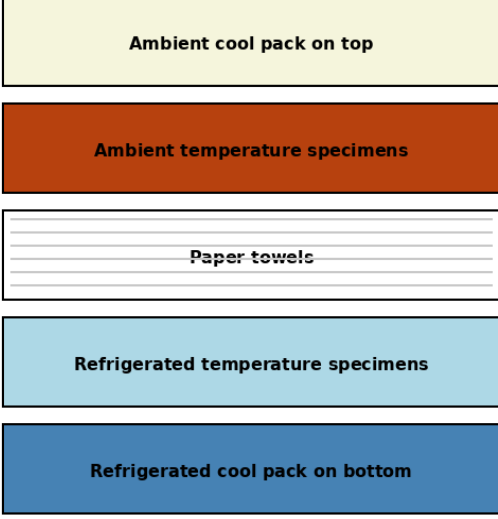


Seasonal instructions for loading lockboxes:

<p>Summer lockbox instructions During warm months (April to October) specimens should be loaded into the lockbox with insulation to prevent overheating.</p> <ul style="list-style-type: none"> • Place a frozen cool pack in the bottom of the lockbox • Use three or four paper towels to layer over the frozen cool pack • Place refrigerated specimens on top of the paper towels • Use three or four paper towels to layer over the refrigerated specimens • Place ambient temperature specimens on top of the paper towels 	<p>Winter lockbox instructions During cold months (November to March) specimens should be loaded into the lockbox with insulation to prevent freezing.</p> <ul style="list-style-type: none"> • Place a refrigerated cool pack in the bottom of the lockbox • Place refrigerated specimens on top of the cool pack • Use three or four paper towels to layer over the refrigerated specimens • Place ambient temperature specimens on top of the paper towels • Place non-refrigerated /ambient cool pack on top of the ambient temperature specimens 
--	--

***Note- The Lab isn't responsible for providing any of the insulating materials and the courier doesn't replace them.**